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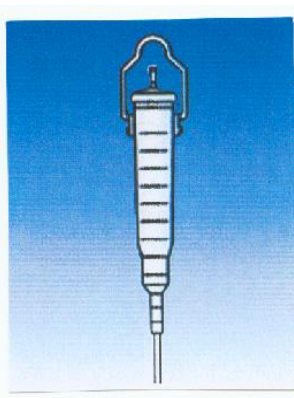
REF 776150

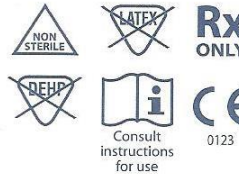
Kangaroo™

**Epump and Joey Burette
Recertification Set**

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en
Directions:

1. Power down pump if on.
2. Fill Recertification Burette to top of reference line as indicated with 50 mL of water.
3. Hang Recertification Burette with fill mark exactly 18" (45.7cm) above the top of pump. For use with the Kangaroo™ Epump enteral feeding pump, the bottom of the feeding container must be 18" above the top of the feeding pump. For use with the Kangaroo™ Joey enteral feeding pump, the fluid column in the feeding container must be 6" above the top of the feeding pump.
4. Place outlet connector into drain at level of bottom of pump.
5. Open blue door enclosing feeding set loading area.
6. To load feeding set:
 - Grasp thumb tab on valve and insert into left pocket.
 - Grasp black ring retainer and gently wrap tubing around rotor and insert retainer into right pocket.
 - Press down on thumb tab to ensure valve is fully seated.
7. Close blue door.
8. Press **POWER** button on control panel.
Note: Display should read **MANUAL TEST**.
9. Individually select each menu item and follow screen prompts for that item.
Note: Each of the first 4 menu items must be successfully completed prior to **START AUTO TEST**.
Note: During **MISTIC TEST** lift black ring retainer Slowly from its pocket. Listen for tone and release retainer. If unsuccessful, repeat operation.
10. Upon completion of auto test procedure display will indicate whether pump has passed or failed re-certification.
11. Consult Pump Operation and Service Manual for further details.

Warning: Contains small parts. Keep out of reach of children.

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© 2012 Covidien. Made in Mexico.